# Aseptol S.A.

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# Aseptol S.A.

## Cleaning and disinfection of aspiration systems

---

### Composition

Quaternary ammonium, polyhexanide, non-ionic detergent, corrosion inhibitor, sequestrant.

### Properties

ASEPTOL S.A. is formulated without aldehyde.  
ASEPTOL S.A. is designed to clean aspiration ducts (saliva ejectors and other surgical aspiration devices) of saliva, blood, calculus and any organic debris, that can be aspirated from the oral cavity.  
It is formulated to disinfect both tubes and filters.  
ASEPTOL S.A. also deodorises all the aspirating system. It is non corrosive and does not damage the engine. It does not promote foam formation.

### Instructions for use

Prepare a 2% ASEPTOL S.A. solution (i.e. one 20 ml dose per litre of water). Aspirate the solution into the aspiration system.

### Warnings and precautions for use

For external use only. Do not swallow.  
Xi - Irritant. Contains polyhexamethylene biguanide hydrochloride. May cause allergic reactions.  
Harmful to aquatic organisms, may cause long-term adverse effects in the aquatic environment.  
Irritating to skin.  
Risk of serious damage to eyes.  
In case of contact with eyes, rinse immediately with plenty of water and seek medical advice.  
Wear suitable protective clothing and eye/face protection.  
In case of accident or if you feel unwell, seek medical advice immediately (show the label where possible).  
Wear suitable gloves.  
Avoid release to the environment. Refer to special instructions/safety data sheets  
Keep out of the reach of children.

### Microbiological properties

Bactericidal: NF EN 1040 (5 min), NF T 72-170 (spectrum 4)  
Active against *Candida albicans* as per NF EN 1275 (5 min).  
Active against HIV-1 and HBV (Hepatitis B) viruses.

### Presentation

5-litre plastic bottle with dispensing pump (20 ml per actuation).

*For professional use only in the practice of dentistry.*

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