FR

**COMPOSITION**

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Disodium edetate, cetrimide, excipient  
(pH: 6.5 to 7.5)

**PROPERTIES**

Disodium edetate dissolves the mineral content of root canals and facilitates mechanical preparation of narrow canals or canals difficult to access. It also helps to remove pulp stones.

Because of its very low surface tension (39.7 dynes per centimeter), the product easily penetrates the narrowest canals and softens the walls.

Due to its balanced pH, it is well tolerated by the periapex.

**INDICATIONS**

Locating canal orifices.  
Facilitating root canal preparation.  
Aiding in the removal of pulp stones.

**INSTRUCTIONS FOR USE**

Place the solution in the pulp cavity with a pipette and use a fine dental probe to make it flow along the root canal.

The canal is then ready for mechanical preparation.

Repeat several times. Rinse with water.

For pulp stones: soak a cotton pellet in the solution, place it in the pulp chamber, cover with a temporary cement and leave the product to act for a few days.

**SPECIAL WARNINGS AND PRECAUTIONS FOR USE**

Xi – Irritating to eyes and skin.

In case of contact with eyes, rinse immediately with plenty of water and seek medical advice.

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