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## EN Sterile siliconised dental needles

### DIRECTIONS FOR USE

- Break the sterility seal just before use by turning the two parts of the protective sheath in opposite directions.
- Completely separate the two parts of the protective sheath by removing the sheath cap to reveal the end of the needle which will be fixed onto the syringe.
- Screw the plastic nozzle of the needle onto the threaded hub of the syringe by holding the second part of the sheath which protects the other end of the needle.
- Remove the plastic protective sheath with a slight twisting motion just before the injection is administered. (It can be used again when removing the needle from the syringe to avoid the risk of needlestick injury).
- Carefully unscrew the needle from the syringe using the plastic protective sheath or any other means to protect you from needlestick injury.

Discard the used needle into a puncture-resistant container in accordance with all applicable medical waste regulations.

### WARNINGS

Do not re-use the same needle on other patients. Due to needle point damage on initial insertion, it recommends to replace the needle for each cartridge!

Extreme physical pressure or strength may cause the needle to break. Avoid extreme or repeated movements of the needle during injection as this may cause the needle to break.

### DO NOT RECAP NEEDLES BY HAND.

Do not bend, break, or otherwise manipulate needles by hand. Do not exceed the expiry date shown on the outer packaging. Failure to follow warnings is the full responsibility of the user.

### PRESENTATION

100 two-point disposable non-sterilisable dental needles

*For professional use only in the practice of dentistry.*

### (GB) - Distributor :

SEPTODONT L.t.d.

Units R & S - Orchard Business Centre

St Barnabas Close

Allington, Maidstone, Kent ME 16 0UZ - UK

### (AU) - Sponsor : Specialities Septodont Pty Ltd

14 Cliffbrook Crescent, Emlu Plains, NSW 2750

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